# FRANÇAIS

## 1. UTILISATION

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## ENGLISH

### 1. INTENDED USE

Respiro Hospital F2 Vented Disposable is an oronasal mask with calibrated exhalation orifices intended to be used for short time periods (less than 7 days) in a clinical setting by a single adult patient (weighing over 30kg), who has been prescribed for a non-invasive positive pressure ventilation therapy (NIPPV), e.g. continuous positive airway pressure therapy (CPAP) or Bilevel positive airway pressure therapy (BiPAP).

### ⚠️ WARNINGS

- The mask should be used exclusively with positive pressure equipment recommended by your doctor or respiratory therapist supplying a minimum pressure of 4 cmH<sub>2</sub>O: at lower pressures, the guaranteed air safety evacuation through the exhalation orifices is not guaranteed and so a partial re-inhalation may occur.
- The anti-asphyxia valve fitting ensures patient safety even in case of failure of the positive pressure equipment. Do not use the mask if the valve is damaged or distorted.
- Before using the mask, it needs to be checked that the exhalation orifices are not occluded, even partially, to not jeopardize the safety and the quality of the therapy.
- In case of discomfort, irritation or allergic reactions to any component of the mask, consult your doctor or respiratory physiotherapist.
- In case of additional oxygen administration, smoking or using open flames is prohibited.
- When using oxygen and the equipment is not in operation, turn off the oxygen dispenser.
- Do not use the mask in case of vomiting or nausea.
- Keep away from light.
- Before use, check the integrity of the mask. If damaged during transport, notify your dealer.
- Do not leave the components of the mask unattended, some of these may be swallowed by children.
- The device is a single use device: the reuse on multiple patients may result in transmission of infections serious harm or even the death of the patient.
- Contact your dealer as a healthcare professional in case of functional problems, size problems or questions regarding the application of the medical device.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

### 2. TROUBLESHOOTING

PROB-LEMS	POSSIBLE CAUSES	SOLUTIONS
The airflow doesn't reach the mask	- Flow generator turned off or not working - Breathing circuit ventilator not inserted correctly	- Switch on or adjust the flow generator - Connect the breathing circuit ventilator to the flow generator and the mask
Noisy and annoying air leaks	- The mask has not been assembled correctly - The mask is not positioned correctly	- Disassemble the mask and then reassemble it as described in the user manual - Reposition the mask on your face and adjust the headgear straps as described in the user manual
	- The size of the mask may not be correct	- Contact your doctor or therapy technician to find the right size

### 3. DISPOSAL

The mask components are made of the materials indicated below. They do not contain hazardous substances and can therefore be disposed of with ordinary household waste.

COMPONENT	MATERIAL
Cushion (a)	Silicone
Frame (b)	Polycarbonate
Forehead support (c)	Silicone
Anti-asphyxia valve fitting (d)	Polycarbonate + Silicone
Swivel fitting (e)	Polycarbonate
Headgear (f)	Nylon/UBL/Polyurethane
Clips (g)	Nylon

### 4. TECHNICAL DATA PRESSURE-FLOW CURVE: Refer to section B.

Regulatory information	CE marking in accordance with Regulation 2017/745 (EU) and subsequent implementation amendments. Class IIa Main product standard UNI EN ISO 17510
------------------------	---

Operating pressure range	4 cmH <sub>2</sub> O / 32 cmH <sub>2</sub> O
Operating temperature range	+5 °C / +40 °C
Dead space	S: 285 ml - M: 295 ml - L: 326 ml
Resistance	à 50 l/min: 0,20 cmH <sub>2</sub> O - à 100 l/min: 0,33 cmH <sub>2</sub> O
Resistance in single fault condition, when the anti-asphyxia valve is opened in the atmosphere	Inhalation at 50 l/min: 0,7 cmH <sub>2</sub> O Exhalation at 50 l/min: 0,5 cmH <sub>2</sub> O
Opening pressure of the anti-asphyxia valve	0,55 cmH <sub>2</sub> O
Closing pressure of the anti-asphyxia valve	0,95 cmH <sub>2</sub> O
A-Weighted Sound Pressure (UNI EN ISO 3744)	33,5 dB(A)
A-Weighted Sound Pressure (UNI EN ISO 3744)	41,5 dB(A)
Dimensions (without headgear)	S: 200x100x115 mm - M: 200x105x115 mm - L: 210x100x115 mm
Weight	S: 137 g - M: 139 g - L: 140 g

### 5. WARRANTY CERTIFICATE

Air Liquide Medical Systems S.r.l. guarantees its product against material or manufacturing defects for 180 days from the date of purchase, provided that the conditions of use in the instructions for use are observed. If the product is faulty under normal use, Air Liquide Medical Systems S.r.l. will, at its discretion, repair or replace the faulty part or its components. The manufacturer shall be responsible for the safety, reliability and performance of the device if it is used in accordance with the operating instructions and its intended use. Keep the original casing for the entire warranty period. The warranty in case of product defects can be applied by contacting your dealer. For more information on the rights under the warranty, please contact your retailer.

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